Distributor Details:

lumie[®]

Outside In (Cambridge) Ltd.

3 The Links,
Trafalgar Way,
Bar Hill,
Cambridge,
CB23 8UD, UK

www.lumie.com

LUMIE is a registered trademark of
Outside In (Cambridge) Limited.

The Mini Luminaire is a registered design
of Outside In (Cambridge) Limited.

© Lumie 2023
24/08/2023
1-782-000003-01

ENG

Scan this QR code to download the full user guide, or type the following link into your web browser:
<https://www.lumie.com/en-us/support/product-guides/mini>

Safety

WARNING - To reduce the risk of fire, electric shock or injury:

Read instructions for use.

If the unit is damaged, contact Lumie before use.

Keep away from water and damp.

For indoor use only.

The unit gets warm when the light is on, so make sure that it is not covered.

Use only the mains power adaptor supplied.

Keep the mains power adaptor and cable away from heated surfaces. If either are damaged, they should be replaced with an approved Lumie product.

Only to be used by adults and by children when supervised.

Do not attempt to service or modify any part of the unit.

If the unit has been stored outside the recommended operating temperature range (5 °C to 35 °C / 41 °F to 95 °F), allow it to stand for at least 1 hour before applying power to avoid condensation or overheating.

The light source contained in this luminaire shall only be replaced by Lumie or an authorized Lumie distributor.

Not to be used by or on individuals who are unconscious, anaesthetized or incapacitated e.g. under the influence of alcohol.

CAUTION

Do not drop the unit on the floor, hit it hard or expose it to other shocks.

Place the unit on a stable, level and non-slippery surface.

Always unplug the unit before cleaning and allow it to cool down.

Not to be used with light dimmers, timers, motion detectors, voltage transformers or extension cords.

Consult your doctor before you start using Mini if you have suffered from or are suffering from serious depression.

Intended for household use, including similar use in hotels. Do not use attachments not recommended by Lumie.

Disclaimer

It is not possible for Lumie to warn the consumer about every possible danger related to installation and use of the product. The consumer must use his or her own common sense and good judgment when installing and using the product.

Electromagnetic compatibility

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This Class B digital apparatus complies with Canadian ICES-005.

For further detailed information on the electromagnetic environment in which the device can be operated refer to the website: www.lumie.com/pages/e-m-c.

Responsible Party Address

Violette Engineering Corporation
313 Park Avenue Suite 300
Falls Church, VA 22046 USA
info@violettecorp.com

Disposal

Do not throw out the mains power adaptor or the unit with normal domestic waste. Take them to an official recycling point to be disposed of responsibly.

Precautions

Research shows that light therapy is very safe. However, please check with your doctor first if any of the following apply:

- You have a major eye problem or have had eye surgery;
- You have been advised to avoid bright light or are taking medication that may cause photosensitivity;
- You have been diagnosed with depression or other mood/sleep disorders. This is especially important if you are also taking medication for this, since light therapy can affect your mood.

A few people experience side effects but these are generally mild (including headaches, eye strain, nausea and hyperactivity) and can usually be solved simply by switching off the light. Over the next few days, gradually build up the time you spend in front of it to find a treatment time that works best for you.

Some people complain of difficulty sleeping and often this is down to using the light too close to bedtime.

If you have any problems while using Mini, or if you've been using the light for a week but haven't noticed any benefit, please contact Lumie or speak to your doctor.

IMPORTANT: the guidelines in this leaflet are not medical directions for treating any condition.

Warranty

This unit is covered under warranty by Lumie LLC against any manufacturing defect for 3 years from the date of purchase. If you need to return the product for servicing, use the original packaging (including internal packing) if possible. Make sure the mains power adaptor is prevented from moving in transit and damaging the unit. If the unit is received damaged, through misuse or accidental damage, then we may have to charge to repair it; we will contact you first. Please contact Lumie for further advice.

For all warranty details, please refer to our Limited Warranty which can be found at <https://www.lumie.com/en-us/warranty>.

SN Serial number (see back of unit)

Technical description

Subject to change without notice.

Lumie Mini

Detachable accessory: Mains power adaptor (supplied)
Rated Input: 100–240 V AC, 50/60 Hz, 0.4 A max.
Rated Output: 12.0 V = 0.85 A (12.0 W)

Operating conditions:

Temperature: 5 °C to 35 °C / 41 °F to 95 °F
Relative humidity: 15% to 93% (non-condensing)
Atmospheric pressure: 70 kPa to 106 kPa

Storage/transport conditions:

Temperature: -25 °C to +70 °C / -13 °F to +158 °F
Relative humidity: 15% to 93% (non-condensing)
Atmospheric pressure: 25 kPa to 106 kPa

The weights, dimensions, capacities, performance ratings, characteristics and other data displayed are mentioned only as general information. They shall not bind Lumie.

Online guide

You can view or download an online copy of the latest version of this guide on the appropriate product page of the Lumie website.

Lumie retains the right to modify the guide as it deems appropriate.

The consumer is charged with the knowledge of the modifications set forth in the guide.

Contact us

If you have any questions or problems regarding this product, or have suggestions that you feel could help us improve our products or service please email the Lumie Careline at info@lumie.com.

Distributed in the US and Canada by:
Lumie LLC, 4600 140th Avenue N., Suite 180, Clearwater, FL 33762.

Outside In (Cambridge) Limited, 3 The Links,
Trafalgar Way, Bar Hill, Cambridge, CB23 8UD, UK.

Lumie is a registered trademark of:
Outside In (Cambridge) Limited, incorporated in England and Wales.
Registered Number: 2647359.

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Outside In (Cambridge) Limited, 3 The Links, Trafalgar Way, Bar Hill, Cambridge, CB23 8UD, UK.
Outside In (Cambridge) Limited, constituida en Inglaterra y Gales.