## Distributor Details:

This unit is covered under warranty against any manufacturing defect for 3 years from the date of purchase. This is in addition to your statutory rights. If you need to return the product for servicing, use the original packaging (including internal packing) if possible. Make sure the mains power adaptor is prevented from moving in transit and damaging the unit. If the unit is received damaged, through misuse or accidental damage, then we may have to charge to repair it; we will contact you first. Please contact Lumie for further advice.

	Serial number (see base of unit)		Importer
--	----------------------------------	---	----------

## Technical description

Subject to change without notice.

**Lumie Dash**  
Detachable accessory: Mains power adaptor (supplied)   
Rated Input: 100–240 V AC, 50/60 Hz, 0.6 A max. (PF ≥ 0.9)  
Rated Output: 12 V = 2 A (24 W)

Operating conditions:  
Temperature: 5 °C to 35 °C  
Relative humidity: 15% to 93% (non-condensing)  
Atmospheric pressure: 70 kPa to 106 kPa  
Storage/transport conditions:  
Temperature: -25 °C to +70 °C  
Relative humidity: 15% to 93% (non-condensing)  
Atmospheric pressure: 25 kPa to 106 kPa

Colour temperature: 5000 K  
Colour rendering index: 95

At maximum brightness, Dash delivers 10000 lux at 15 cm.  
This product contains a light source of energy efficiency class F.

	European Declaration of Conformity		UK Conformity Assessed
--	------------------------------------	---	------------------------

## Contact us

If you have any questions or problems regarding this product, or have suggestions that you feel could help us improve our products or service please email [info@lumie.com](mailto:info@lumie.com).

 Outside In (Cambridge) Limited, 3 The Links, Trafalgar Way, Bar Hill, Cambridge, CB23 8UD, UK.

Lumie is a registered trademark of: Outside In (Cambridge) Limited, incorporated in England and Wales. Registered Number: 2647359.

VAT No:  
GB 880 9837 71 (UK)  
NL 826301381B01 (EU)

© Lumie 2024  
26/03/2024  
1-782-000004-01

## ENG

Scan this QR code to download the full user guide, or type the following link into your web browser:  
<https://www.lumie.com/support/product-guides/dash>

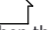
## Safety

Read instructions for use. 

If the unit is damaged, please contact Lumie before use.

Keep away from water and damp. 

For indoor use only. 

The unit gets warm when the light is on, so make sure that it is not covered.

Use only the mains power adaptor supplied.

Keep the mains power adaptor and cable away from heated surfaces. If either are damaged, they should be replaced with an approved Lumie product.

This appliance does not have a ON/OFF switch. To power down or reset, remove the mains power adaptor from the wall socket. You should therefore ensure the socket is easily accessible.

Only to be used by adults and children over 7 years old. Children using the light, or playing near it, should be supervised.

Caution – do not attempt to service or modify any part of the unit. 

If the unit has been stored outside the recommended operating temperature range (5 °C to 35 °C), allow it to stand for at least 1 hour before applying power to avoid condensation or overheating.

The light source contained in this luminaire shall only be replaced by Lumie or an authorised Lumie distributor.

### Electromagnetic compatibility

Electrostatic discharge may affect the working of this product. If this occurs, simply switch off for at least 10 seconds and the unit will reset.

Dash uses a touch sensor to control the light. In rare situations, external conditions may trigger the touch sensor. This can occasionally cause the light to vary in intensity or turn on or off. If this happens, use the touch interface to set the light to the previous level. Dash will continue to operate normally and remains safe to use.

### Disposal

Do not throw out the mains power adaptor or the unit with normal domestic waste. Take them to an official recycling point to be disposed of responsibly. 

### WARNING!

Check with your doctor before using this product if any of the following apply:

You have a major eye problem or have had eye surgery;

You have been advised to avoid bright light or are taking medication that may cause photosensitivity;

You have been diagnosed with depression or other mood/sleep disorders. This is especially important if you are also taking medication for this, since light therapy can affect your mood.

## Warranty

This unit is covered under warranty against any manufacturing defect for 3 years from the date of purchase. This is in addition to your statutory rights. If you need to return the product for servicing, use the original packaging (including internal packing) if possible. Make sure the mains power adaptor is prevented from moving in transit and damaging the unit. If the unit is received damaged, through misuse or accidental damage, then we may have to charge to repair it; we will contact you first. Please contact Lumie for further advice.

	Serial number (see base of unit)		Importer
---	----------------------------------	---	----------

## Technical description

Subject to change without notice.

**Lumie Dash**  
Detachable accessory: Mains power adaptor (supplied)   
Rated Input: 100–240 V AC, 50/60 Hz, 0.6 A max. (PF ≥ 0.9)  
Rated Output: 12 V = 2 A (24 W)

Operating conditions:  
Temperature: 5 °C to 35 °C  
Relative humidity: 15% to 93% (non-condensing)  
Atmospheric pressure: 70 kPa to 106 kPa

Storage/transport conditions:  
Temperature: -25 °C to +70 °C  
Relative humidity: 15% to 93% (non-condensing)  
Atmospheric pressure: 25 kPa to 106 kPa

Colour temperature: 5000 K  
Colour rendering index: 95

At maximum brightness, Dash delivers 10000 lux at 15 cm.  
This product contains a light source of energy efficiency class F.

	European Declaration of Conformity		UK Conformity Assessed
---	------------------------------------	---	------------------------

## Contact us

If you have any questions or problems regarding this product, or have suggestions that you feel could help us improve our products or service please email [info@lumie.com](mailto:info@lumie.com).

 Outside In (Cambridge) Limited, 3 The Links, Trafalgar Way, Bar Hill, Cambridge, CB23 8UD, UK.

Lumie is a registered trademark of: Outside In (Cambridge) Limited, incorporated in England and Wales. Registered Number: 2647359.

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Outside In (Cambridge) Limited, registrert i England og Wales.